

Wrist Splint Neutral Position Stabilizer: Universal



The ISO Preferred Wrist Splint Neutral Position Stabilizer: Universal is a neutral position stabilizer that reduces the time it takes to heal discomfort from injuries or post-surgery. Please follow your orthopedic professional's instructions explicitly and adjust or discontinue use only as directed to achieve maximum benefit for your concerns.

ISO Preferred Wrist Splint Neutral Position Stabilizer: Universal (WR301) can be worn on either hand and is a splint that provides neutral position stabilization and is intended for the prevention of finger and wrist contraction while assisting with flaccid paralysis, soft tissue contracture management, nerve injury, arthritis, and the support of weak or damaged parts of the forearm. With its unique ROM cuff hinge is excellent for wrist/hand ailments that may require flexion/extension control. Also intended for surgical repair and after cast management.

1. Disengage all fastening straps and connections. Open and separate the inner adjustable material to a wide enough space that your hand and wrist are easily able to slide into the brace.
2. Slide the brace over affected wrist/hand. Be sure your thumb and thumb hole in the compression fabric are on the same side of the brace with the fastening straps and your palm facing the ceiling. Once your thumb is securely in place, use your other hand to squeeze the rear and then the front of the malleable materials equally over your forearm. Your forearm and wrist should be comfortable in the brace.
3. Tighten and secure the two fastening straps closest to your elbow first. Adjust the hand portion to the size of your fingers width-wise, with your fingers softly touching each other. Now rotate the hand portion of the brace into your doctor's instructed location, and secure into place by rotating the wheel clockwise until its tight.
4. Take hold of the last remaining fastening strap and pull it over your wrist on a slight downward angle (4a), then cross back over the lower portion of your thumb and secure the fastening strap into place (4b).
5. Your hand, wrist, and forearm should all be centered appropriately inside the brace. The brace should feel snug and secure. If it is too tight or loose at any point, please adjust the straps accordingly. **DO NOT OVERTIGHTEN.**





INTRODUCTION

Your **ISO Preferred Wrist Splint Neutral Position Stabilizer: Universal (WR301)** is designed to provide your body with the support and stability that it needs to successfully manage or eliminate your mobility concerns. It is critical that you follow your orthopedic professional's directions closely to ensure that the apparatus is worn properly and in the manner that will be most beneficial in addressing your issues. For correct application, please read the enclosed instructions and watch the on-line instructional video (if available) at www.ISOpreferred.com so that this device can provide you with superior long-term usability and care.

USAGE RECOMMENDATIONS

It is vital that you read the included instructions and when applicable watch the instructional video for the **ISO Preferred Wrist Splint Neutral Position Stabilizer: Universal (WR301)**. The brace is intended for application by qualified professionals only. It is not recommended for patients to fit or apply or adjust their new brace without an orthopedic professional's expertise, unless otherwise instructed. Further questions regarding adjustments or changes, frequency of use, and/or duration of use should be presented to your orthopedic professional, as they are familiar with the specific needs and conditions, for which this device was prescribed.

CARE AND CLEANING INSTRUCTIONS

Please do not place your brace in the washing machine or dryer at any time. When preparing to clean your **ISO Preferred Wrist Splint Neutral Position Stabilizer: Universal (WR301)** you will want to separate the parts where applicable and care for them individually.

- Do not dry clean, iron, bleach, or use an abrasive cleaning instrument on any part of the brace. Failure to follow these instructions may damage your brace and void the manufacturer's warranty and it may no longer work properly.
- Hand wash each section in cold water with a mild detergent. Lay flat to air dry.
- Allow all parts to air dry prior to reassembling the device.

CAUTION

The **ISO Preferred Wrist Splint Neutral Position Stabilizer: Universal (WR301)** is for single patient use only. While every effort has been made to obtain maximum strength, function and durability, there is no guarantee that injury will be prevented through the use of this device, especially if the instructions are not followed correctly.

DO NOT OVERTIGHTEN.